



# MGC Health<sup>TM</sup> Raised Toilet Seat with Lock & Arms in Microban<sup>®</sup>

POWERED BY  
**MICROBAN**<sup>®</sup>

**MGC HEALTH<sup>TM</sup> Raised Toilet Seat with Lock & Arms MICROBAN** antimicrobial and antifungal technology is an essential bathroom aid accessory designed to enhance safety and accessibility. This product ensures a secure and comfortable experience for individuals who may face challenges standing or sitting on a standard toilet. The Raised Toilet Seat provides stability and support. MGC Health prioritizes the quality and functionality of our Bath Safety Products, manufactured at top-tier facilities and subjected to thorough safety and durability testing by Bureau Veritas. SGS inspections guarantee the excellence of each shipment. Like all MGC Health products, our Raised Toilet Seat comes with a limited lifetime warranty, offering unparalleled value and peace of mind.

## Key Product Features

1. Protected by MICROBAN Antimicrobial & Antifungal Technology that helps prevent the growth of damaging stains and odor-causing bacteria on the product
2. Weight Capacity: 300 lb, Strongly Built with durable and corrosion proof Aluminum frame and plastic with side arms that provides additional stability
3. No assembly required. Elevates the toilet by 5"
4. Lightweight bath seat is easily movable and provides comfort and safe experience while bathing to help seniors, elderly, and disabled
5. Built in locking mechanism provided additional stability and support



## Product & Packaging Specification

Specification	Description
UPC	383173000771
MGC Ref	IT000096
Material	HDPE
Weight Capacity	300 lbs
Warranty	Limited Lifetime
Retail Box Shelf dimensions	15.4. x 6.3 x 15.2
Shipping Dimensions	14 x 16.1 x 34.5
Units per Case	4
Cases per pallet	54

## Assembly Instruction

### A. 1 Seat riser

1. Lift the toilet seat and lid. Note: certain bowl types may require the removal of the existing seat and/or lid to ensure proper installation.
2. Place the toilet seat riser on the toilet rim such that the seat risers rests flat on the toilet rim.
3. Center the seat riser on the bowl rim and push it back as close to the inner rim of the toilet such that the riser sits flat on the top of the toilet lip.  
Note: If the rear topside of the riser is hitting the toilet seat and/or lid, remove the seat and lid.